

Urine collection bag
Leg bag with integrated strap



Instructions for use

23315232 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved.© (2013-11-25)

Indication

Urine collection bag

Information

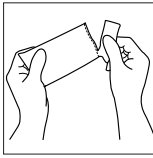
The bag has a capacity of 250mL and is for daytime use. The bag is designed for use with urisheaths. Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

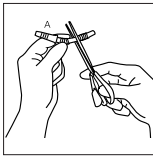
Warning

Re-use of the single use product may create a potential harm to the user. Reprocessing, washing, disinfection, and sterilisation may compromise product characteristics, causing additional risk of physical harm to or infection of the user.

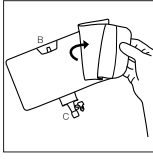
How to use Conveen Active

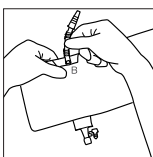
Fitting the leg bag (right leg)

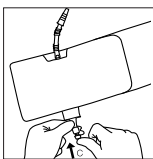
- 1  1. Open the packaging by tearing along the perforation. Remove the leg bag and the small piece of separate tubing.

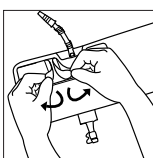
- 2  2. The tubing has smooth and corrugated sections. At one end, the inlet connector (A) is attached.

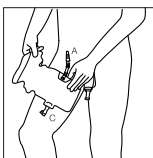
The length of the tube may be shortened by half if preferred. To shorten the tube, simply cut the free end with scissors immediately after the first corrugated section of tubing. It is important to retain as much of the subsequent smooth section as possible. This will help create a secure connection between leg bag and tube when mounting the tube.

- 3  3. Unfold the leg bag and identify the inlet valve at the top of the bag (B) and the outlet tap at the bottom of the bag (C).

- 4  4. Attach the free end of the tube to the inlet valve (B).

- 5  5. Close the outlet tap (C).

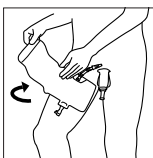
- 6  6. Remove the protective film from the adhesive patch on the backside of the collection bag.

- 7  7. Hold the bag with the inlet connector (A) pointed upwards and the outlet tap (C) pointed downwards. The adhesive patch should be facing your right thigh.

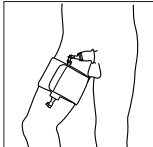
Attach the bag directly to the skin of your inner right thigh.

When positioning the bag, make sure it is placed high on your thigh with the inlet connector centred on your inner thigh, immediately below the penile area. The aim is a position of the bag that allows for comfortable and secure connection to a urisheath.

When first fitting the bag, it may be necessary to reposition it a few times to discover the optimal position for you.

- 8  8. Wrap the long section of strap around your leg and use the burr fabric to fasten the end of the strap.

The fit should be comfortable and firm. It is not necessary to have the strap wrapped very tightly around your thigh. The adhesive patch will help hold the bag securely in place.

- 9  9. Connect the bag to a urisheath by pushing the bag's inlet connector firmly into the urisheath outlet.

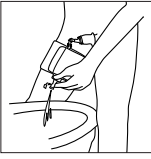
10



10. For comfortable wear and discretion, hide the outlet tap by folding it upwards and tucking it into the pocket between the bag and the outer layer of elastic strap material.

Emptying and disposal

11



11. The leg bag should be emptied before it becomes completely full.

Untuck the outlet tap and position it over a toilet bowl or other suitable receptacle.

Grasp the outlet with one hand. Squeeze the flexible part between your fingers. With the other hand, open the tap. Ease the pressure on the flexible part of the outlet tap gradually to allow the urine to flow.

Always remember to close the tap after emptying the bag.

Return the tap to the pocket under the elastic strap material.