Instructions for Use

23319628 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2017-05-15

Distributed by / Distribué par
Coloplast Canada Corporation
1380 Creditstone Road, Unit 6&7
Concord, ON L4K 0J1
Tel.: 1-888-880-8605

Indication

Urinary catheter for intermittent use.

Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

The catheter can be exposed to extreme temperatures (below 0°C and up to 60°) for up to 24 hours without the catheter being damaged.

This product is a coated catheter in sterile solution with attached bag.

The solution in which the catheter is stored is harmless. However, the solution may stain.

Caution

Contact your health care professional before you perform self-catheterization for the first time.

Catheterization should take place as quickly as possible after opening the package. The catheter should always be removed when the bladder has been emptied.

Contact your health care professional if you:

- experience symptoms such as fever, discomfort when emptying the bladder, a frequent urge to urinate or blood in the urine.
 - have been diagnosed with urethral strictures.
 - experience that the catheter does not drain the bladder the way you are used to.
- Do not use if package is damaged.

Keep away from sunlight

Store at room temperature and at dry conditions

Should be stored in a horizontal position.

Warning


Re-use of the single use product may create a potential harm to the user. Reprocessing, washing, disinfection, and sterilization may compromise product characteristics, causing additional risk of physical harm to or infection of the user.

How to use the product


Opening / Preparing the Product


Start by washing your hands and the area around the urethral orifice. It is important to avoid transferring bacteria from your hands to the catheter.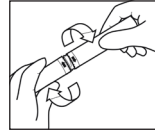

Twist and remove the cover.

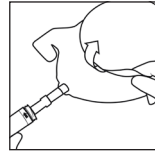
Unfold the bag.

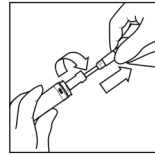
Twist and pull the catheter out.

Insertion of the Catheter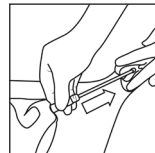



Females: Insert the catheter into the urethra until urine starts to flow.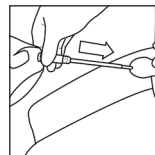


Males: Lift the penis with one hand to straighten out the urethra. Insert the catheter into the urethra until urine starts to flow.

Removal of the Catheter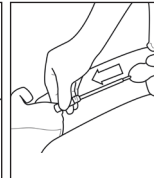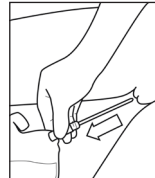


Males + females: When urine stops flowing, pull the catheter out about 1-2 cm. If urine starts flowing again, wait until it stops. Then pull the catheter out 1-2 cm again. When your bladder is empty remove the catheter.


Emptying and disposal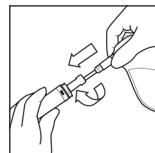

Reseal the catheter by twisting it back into the packaging.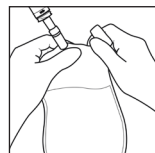

Open the urine bag by tearing off the tap.

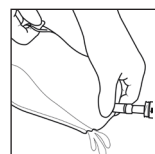Empty the bag into the toilet and dispose the catheter and bag in household rubbish.

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