

Brava®

Elastic Barrier Strips
Elastic Barrier Strip XL
Elastic Barrier Strips -
Y-shape
Elastic Barrier Strips -
Straight



Distributed by:

Coloplast Corp.
1601 W. River Rd. N.
Minneapolis, MN 55411 USA
Tel. (888) 726-7872
www.coloplast.us



Instructions for use

23322484 Version 1
The Coloplast logo is a
registered trademark of
Coloplast A/S. All rights
reserved.©
2019-12-04



Intended purpose

Brava Elastic Barrier Strips are intended to support the rim of the ostomy barrier by providing additional adhesion to the intact peristomal skin.

Indications

Brava Elastic Barrier Strips are indicated for people who use ostomy pouches and barrier/adhesive.

Warnings

Re-use of this single use product is not recommended, cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilization may compromise product characteristics, causing additional risk of physical harm or infection to the user.

Cautions

Keep away from sunlight as it may impact product quality.

Do not stretch the product when applying as it may impact product performance.

Information

Single use product.

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

The device does not have to be removed before an MR scan as it is MR safe.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Special storage conditions

Keep away from sunlight.

Store product horizontally.


How to use

Opening the box

Hold the turquoise tab and pull back to open. Do not tear the tab as it allows you to re-seal the box.

Application


Elastic Barrier Strips Elastic Barrier Strip XL

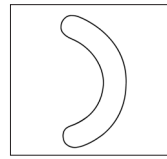


Peel back one piece of protective paper backing to expose the adhesive.

Place the barrier strip where desired on the barrier/adhesive where you need support. Remember that a part of the skin barrier should be covered by the elastic barrier strip.

Do not stretch the product when applying.

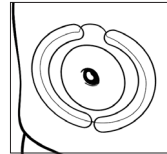


Remove the remaining piece of paper backing and smooth the strip onto the skin. Repeat if necessary.

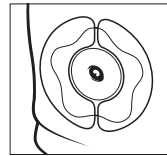

Elastic Barrier Strip XL can also be used with SenSura® Mio Concave barrier.

Application


Elastic Barrier Strips - Y-shape

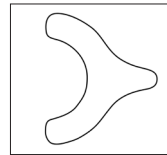

Peel back one piece of protective paper backing to expose the adhesive.

Place the barrier strip where desired on the barrier/adhesive where you need the extra support. Remember that a part of the skin barrier should be covered by the barrier strip.


Do not stretch the product when applying.

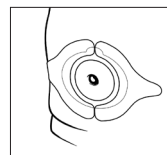

Remove the remaining piece of paper backing and smooth the strip onto the skin. Repeat if necessary.


Application

Elastic Barrier Strips - Straight

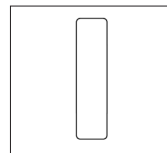

Peel back one piece of protective paper backing to expose the adhesive.

Place the strip where desired on the square barrier/adhesive where you need support. Remember that a part of the adhesive should be covered by the strip. Do not stretch the product when applying.

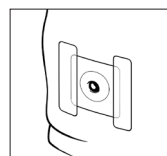

Remove the remaining piece of paper backing and smooth the strip onto the skin. Repeat if necessary.

You can place the strip wherever you feel you need to support the barrier/adhesive.

Removal of the strips

The elastic barrier strip may come off together with the skin barrier. If needed, loosen the edges of the elastic barrier strip and gently stretch the barrier strip for removal.

Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Explanation of symbols



Medical device



Catalog number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Do not re-use



Global Trade Item Number



Keep away from sunlight



This way up



Recyclable packaging



MR Safe