



## How to use

### Fitting the leg bag (right leg)

#### Preparation

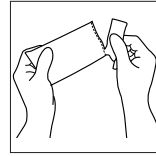


Open the packaging by tearing along the perforation. Remove the leg bag and the small piece of separate tubing.

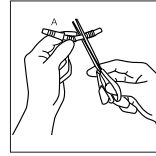


The tubing has smooth and corrugated sections. At one end, the inlet connector (A) is attached.


The length of the tube may be shortened by half if preferred. To shorten the tube, simply cut the free end with scissors immediately after the first corrugated section of tubing. It is important to retain as much of the subsequent smooth section as possible. This will help create a secure connection between leg bag and tube when mounting the tube.

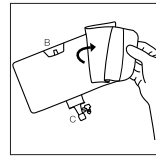


Unfold the leg bag and identify the inlet valve at the top of the bag (B) and the outlet tap at the bottom of the bag (C).

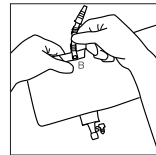

Attach the free end of the tube to the inlet valve (B).

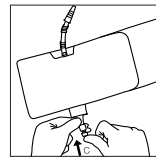

Close the outlet tap (C).

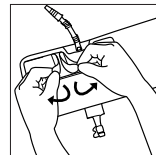

Remove the protective film from the adhesive patch on the backside of the collection bag.



#### Instructions for use

23322864 Version 2

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2020-02-07

#### Distributed by:

Coloplast Corp.  
1601 W. River Rd. N.  
Minneapolis, MN 55411 USA  
Tel. (888) 726-7872  
www.coloplast.us

#### Intended purpose

The urine bag is intended to passively collect urine.

#### Indications

Urine bags are indicated for patients suffering from urinary incontinence and/or retention.

#### Warnings

Re-use of this single use product is not recommended as cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilization may compromise product characteristics, causing additional risk of physical harm or infection to the user.

This leg bag is intended to be used while out of bed and moving about. Change to a bedside drainage bag with higher capacity while in bed and during sleep.

#### Information

The bag is designed for single use.

The bag has a capacity of 8.5 oz./250 mL and is for daytime use.


The bag is designed for use with urisheaths.

The device does not have to be removed before an MR scan as it is MR safe.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

The product is not made with natural rubber latex, however rare contamination with trace amounts of natural rubber latex during manufacturing or packaging may occur.

#### Application

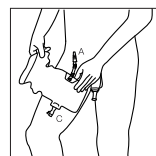


Hold the bag with the inlet connector (A) pointed upwards and the outlet tap (C) pointed downwards. The adhesive patch should be facing your right thigh.

Attach the bag directly to the skin of your inner right thigh.


When positioning the bag, make sure it is placed high on your thigh with the inlet connector centered on your inner thigh, immediately below the penile area. The aim is a position of the bag that allows for comfortable and secure connection to a urisheath.

When first fitting the bag, it may be necessary to reposition it a few times to discover the optimal position for you.

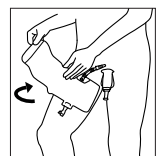


Wrap the long section of strap around your leg and use the burr fabric to fasten the end of the strap.

The fit should be comfortable and firm. It is not necessary to have the strap wrapped very tightly around your thigh. The adhesive patch will help hold the bag securely in place.

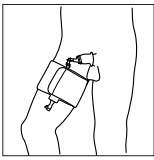


Connect the bag to a urisheath by pushing the bag's inlet connector firmly into the urisheath outlet.

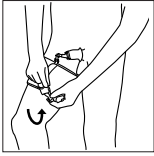

For comfortable wear and discretion, hide the outlet tap by folding it upwards and tucking it into the pocket between the bag and the outer layer of elastic strap material.

#### Removal

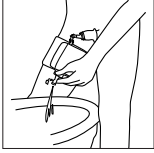

The leg bag should be emptied before it becomes completely full.

Untuck the outlet tap and position it over a toilet bowl or other suitable receptacle.

Grasp the outlet with one hand. Squeeze the flexible part between your fingers. With the other hand, open the tap. Ease the pressure on the flexible part of the outlet tap gradually to allow the urine to flow.

Always remember to close the tap after emptying the bag.

Return the tap to the pocket under the elastic strap material.

#### Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

#### Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

#### Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalog number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Do not re-use



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



Recyclable packaging



Not made with natural rubber latex



This way up