




**Preparation**

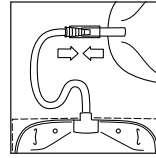


For overnight linkage the Urostomy Night Bag must be fitted with the flexible outlet tube (included in product packaging).

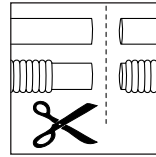


When cutting a corrugated tube, leave as much of the straight section as possible in place. This will ensure a secure connection between tube and connector when mounting the connector.

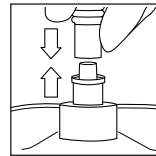

Attach the tube to the bag by pushing the cut end of the tube firmly into the connection inlet of the bag.

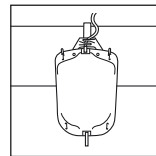


You can use a Conven bag hanger or stand to suspend the bedside drainage bag.

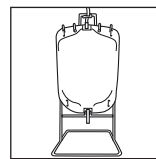

Fit the bag securely on the hanger or stand by inserting the teeth of the hanger or stand in the slits at the top of the bag. Hook the bag hanger onto the bed rail or place the stand on the floor at the bedside.

To ensure optimal drainage, it is important that the bedside drainage bag is positioned lower than yourself when you are lying in bed.

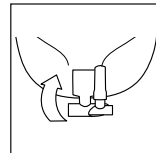


Make sure the outlet tap is closed.

The valve is pointing upwards when it is closed.

Your Urostomy Night bag is now ready to be connected to your urostomy pouch.


2-step outlet

**Application**

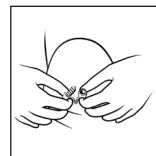


Hold the plug/end of the outlet upwards on your urostomy pouch and squeeze the outer end of outlet, pull out the plug, and hold on the outlet.

You can empty your urostomy pouch before applying it to the Urostomy Night bag.

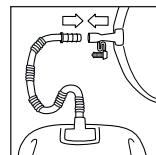

Tube with pre-fitted connector

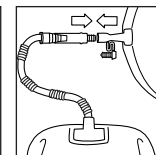

Shortened tube with fitted connectors

Attach the Urostomy Night Bag to the outlet of the urostomy pouch.

The urinary collection system is now functional.



**Instructions for use**

23324118 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2020-08-19

**Distributed by:**

Coloplast Corp.  
1601 W. River Rd. N.  
Minneapolis, MN 55411 USA  
1-800-533-0464  
www.coloplast.us

**Intended purpose**

The Urostomy Night Bag is intended to passively collect urine.

**Indications**

The Urostomy Night Bag is indicated for people suffering from urinary incontinence and/or retention.

**Warnings**

Re-use of this single use product is not recommended as cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilization may compromise product characteristics, causing additional risk of physical harm or infection to the user.

**Information**

The bag is non-sterile and is designed for single use.

The product has a capacity of 2000 mL.

The Urostomy Night Bag can be used together with all Coloplast urostomy pouches.


The night bag's integrated connector attaches securely to the outlet of Coloplast urostomy pouches.

The printed volume indicator is only an indicative volumetric measurement scale.

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

## Emptying

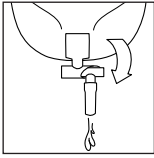

Open and empty the bag in 2 steps:


- 1) Press the valve downwards
- 2) Push on the side for emptying

We recommend emptying the bag when it is a little over half full to ensure optimal drainage.

## Removal

If the Urostomy Night bag is linked to a urostomy pouch, close the urostomy pouch outlet tap. Then disconnect the night bag. Avoid touching the open end of the urostomy pouch outlet tube.

## Disposal

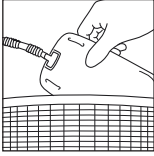

Change the night bag according to local guidelines and recommendations.

Empty the bag before discarding it.

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

## Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

## Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalog number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Do not re-use



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



Recyclable packaging