

SenSura[®] Mio Convex

Urostomy pouch 1-piece

Ostomy Care



Instructions for use

23324372 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2020-10-23

Distributed by:

Coloplast Canada Corp.
1380 Creditstone Road,
Unit 6&7
Concord, ON L4K 0J1
Tel.: 1-888-880-8605
www.coloplast.ca

Intended purpose

The ostomy bag is intended to passively collect output from a stoma. The adhesive baseplate is attached to the ostomy bag and is intended to adhere to intact skin around the stoma.

Indications

The device is indicated for people with a stoma.

Warnings

Re-use of the single use product is not recommended as cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilisation may compromise product characteristics, causing additional risk of physical harm to or infection of the user.

Cautions

For appropriate use, consult your health care professional before you start using a convex product.

Information

Single use ostomy bag.

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

The device does not have to be removed before an MR scan as it is MR safe.


Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Special storage conditions

Store product horizontally.

Keep away from sunlight.


Preparation

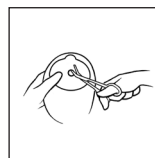

Cut a hole in the base plate to fit the exact size and shape of the stoma using a pair of small curved scissors and the cutting guide on the base plate.

You can use the enclosed stoma measuring guide to measure the size and shape of the stoma.

An exact fit between the hole and the stoma is important to reduce the risk of leakage and skin problems.

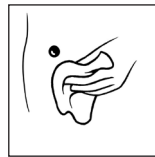


Thoroughly clean the stoma and the surrounding skin area. The skin must be completely dry before application of the base plate.

Application



Peel off the protective film from the base plate.

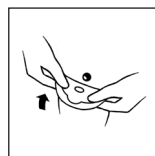


Align the adhesive around the stoma and press it firmly onto the skin.

The adhesive must fit snugly around the stoma. To prevent leakage it is important to ensure a tight seal around the stoma. When applying pressure to the baseplate/bag on the abdomen, begin in the area around the stoma and press the adhesive to the skin. Run a finger all the way around the stoma to ensure that the adhesive seals. Smooth the adhesive down, starting from the stoma and moving outwards to the edges, to ensure that the adhesive has made full contact with the skin.




Apply pressure on the pouch to ensure fast adhesion and a perfect fit between the base plate and the skin around the stoma. Please remember to close the outlet before use.

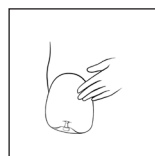

The outlet can be hidden in the pocket on the front to make it more comfortable and discreet.


(Only applies to bags with textile front).

How to empty the pouch

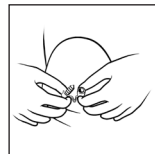

Hold the plug/end of the outlet upwards and squeeze the outer end of outlet, pull out the plug, and relax your hold on the outlet and then drain the pouch. After complete drainage, ensure that the outlet is firmly closed with the plug fully inserted.

Removal

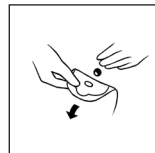


Gently remove the pouch by pulling the removal ear on the base plate away from the stomach and roll the base plate slowly downwards. Apply light pressure on your skin with your free hand to help removal.

Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.


Do not flush the product down the toilet.

Bedside drainage bag

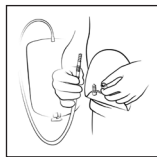

For greater security we suggest you use the Coloplast bedside drainage bag, specially designed to connect directly into your urostomy pouch. However, urostomy pouches can be connected to all types of bedside drainage bags by using the connector supplied. The narrow end of the connector is pushed into the outlet of the urostomy pouch and the other end is connected to the bedside drainage bag.

Belt

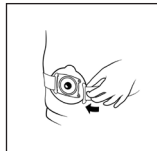

Adjust the belt length by pulling the straps, so it fits your body. There is no need to remove the fabric covering the buckle. Attach each of the four belt hooks to one of the four belt ears on either side of the base plate. Start from the front and apply a light pressure towards the stomach. Then pull the belt hooks away from the coupling, until they “click” in place.

(Only applies to products with belt ears).

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Do not re-use



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



This way up



Recyclable packaging



MR Safe