

Barrier Cream



Coloplast A/S
Hofedam 1
3050 Humlebaek
Denmark

Instructions for use

23325548 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-02-01

Coloplast Limited
Nene Hall
Peterborough Business Park
Peterborough
Cambs. PE2 6FX
Tel: 01733 392000
Fax: 01733 233348

Intended purpose

The Brava Barrier Cream is intended for protection of intact skin around the stoma exposed to intestinal contents, urine and pus.

Indications

Brava Barrier Cream is indicated for people with a stoma.

Cautions

Keep away from sunlight as it may impact product quality.

Information

The product is a barrier cream that allows ostomy adhesives to adhere.

Ingredients

water, cetearyl olivate, sorbitan olivate, tetrahydroxypropyl ethylenediamine, preservative, emulgator

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Special storage conditions

Keep away from sunlight.

How to use

Gently rub a very thin layer onto clean skin. Remove excess cream with a tissue before applying the ostomy adhesive.

Disposal

The product should be disposed of with normal household waste or according to local guidelines. Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Do not re-use



Indicates a barcode as containing Unique Device Identification



Keep away from sunlight



Recyclable packaging