


**SenSura®**  
**SenSura® Xpro**  
**SenSura® Convex**  
**SenSura® Convex Xpro**




**How to use**


**Preparation**

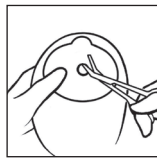

Cut a hole in the adhesive to fit the exact size and shape of the stoma using a pair of small curved scissors and the cutting guide on the adhesive. You can use the enclosed stoma measuring guide to measure the size and shape of the stoma. An exact fit between the hole and the stoma is important to reduce the risk of leakage and skin problems.

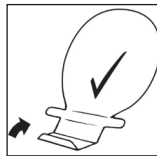

Place the bag on a flat surface with the adhesive side down. Close the outlet by folding the bottom of the outlet upwards three times so that the Velcro® strip becomes visible.

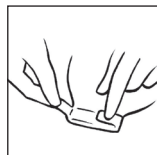

Fold the two tabs inwards to secure the Velcro®.



The outlet can be hidden away by using either the Velcro® dot immediately above the outlet, or by using the Velcro® strip on the middle of the bag.

(This does not apply to the transparent bags where there is no Velcro® strip on the middle of the bag).



Thoroughly clean the stoma and the surrounding skin area. The skin must be completely dry before application of the adhesive.

Ostomy bag 1-piece open

Ostomy Care



**Instructions for use**

23326328 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S.

All rights reserved. © 2021-02-12

Velcro® is owned by Velcro Industries B.V

**Intended purpose**

The ostomy bag is intended to passively collect output from a stoma. The adhesive baseplate is attached to the ostomy bag and is intended to adhere to intact skin around the stoma.

**Indications**

The device is indicated for people with a stoma.

**Warnings**

Re-use of the single use product is not recommended as cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

**Cautions**

For appropriate use, consult your health care professional before you start using a convex product.

**Information**

Single use ostomy bag.

The products are intended to be stored in the box until removed for use, as the box protects the products and the label on the box contains important information like batch code, use-by date and a unique barcode.

The device does not have to be removed before an MR scan as it is MR safe.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

**Special storage conditions**


Store product horizontally.

Keep away from sunlight.

**Application**



Peel off the protective film from the adhesive.

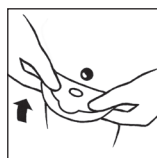


Align the adhesive around the stoma and press it firmly onto the skin. The adhesive must fit snugly around the stoma. To prevent leakage it is important to ensure a tight seal around the stoma. When applying pressure to the baseplate/bag on the abdomen, begin in the area around the stoma and press the adhesive to the skin. Run a finger all the way around the stoma to ensure that the adhesive seals. Smooth the adhesive down, starting from the stoma and moving outwards to the edges, to ensure that the adhesive has made full contact with the skin.



Apply pressure on the bag to ensure fast adhesion and a perfect fit between the adhesive and the skin around the stoma.

### How to empty the bag

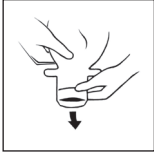

Open the outlet and empty the bag into the toilet when needed.



Clean the outlet, for example with toilet paper. Fold and resecure the outlet after you have emptied the bag.

### Removal



Gently remove the bag by pulling the removal ear on the adhesive away from the skin and roll the adhesive slowly downwards. Apply light pressure on your skin with your free hand to help removal.

### Disposal

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

### Filter label



If required, to prevent gas from escaping, place one of the enclosed filter labels on the filter.

### Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

### Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Do not re-use



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



This way up



Recyclable packaging



MR Safe