


Conveen® Simpla®

Catheter retaining strap




How to use


Fit the catheter retaining strap around your leg or abdomen with the silicone beading against the skin. When fastened, the strap should fit snugly but not too tightly.

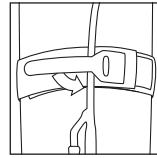

Position the catheter or urine bag tubing under the short silicone fastening strap.

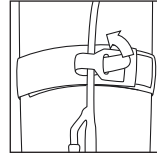

Wrap the short silicone fastening strap around and under the catheter or urine bag tubing and push the free end through the slot.

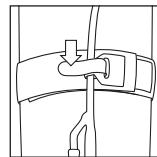

Pull the silicone fastening strap to gently tighten it around the catheter or urine bag tubing. Press the burr-tape fastener into place on the main strap. Do not overtighten.



Coloplast A/S
Hoftevej 1
3050 Humlebaek
Denmark

Instructions for use

23327072 Version 2

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-03-17

Intended purpose

The catheter retaining strap is used to hold a urine bag or catheter in place, by securing it to the leg or abdomen.

Information

The strap may be reused.

The straps can be washed at temperature of 40°C/104°F or lower.

The strap stops the catheter from being accidentally pulled.

Choose the correct length of strap. Cut the strap to the desired length if necessary.

If you experience an allergic reaction, please contact your health care professional.

The product is not made with natural rubber latex, however rare contamination with trace amounts of natural rubber latex during manufacturing or packaging may occur.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

Reusable products

Straps should be replaced when the elasticity weakens.

Make sure the catheter retaining strap fits tightly enough not to slip when you move, but not so tightly that blood circulation is affected.

Laundry and care

Close the catheter retaining strap with the burr-tape fastener and wash according to washing instructions. The strap may shrink up to 5% during washing.

Disposal

Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Not made with natural rubber latex



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



Recyclable packaging