

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

How to use

Cleanse area to be protected, rinse and dry.

Apply Prep Barrier Film and allow to dry forming a protective barrier film on the skin. Slight stinging may be experienced if the skin is denuded.

Not necessary to remove from skin when appliances are changed, as the film will be removed by the adhesive.

Disposal

The product should be disposed of with normal household waste or according to local guidelines.

Do not flush the product down the toilet.

Ingredients

Isopropyl alcohol, Butyl ester of PVM/MA copolymer, Sucrose acetate isobutyrate.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.



Instructions for use

23327206 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2021-03-12

Intended purpose

Protects skin from adhesive trauma and irritation from urine and/or faeces.

Warnings

Reuse of this single use product may create a potential risk to the user. Reprocessing, cleaning, disinfection, and sterilization may compromise product characteristics which in turn create an additional risk of physical harm to or infection of the patient.

Cautions

For external use only.

Do not use near eyes. Will cause eye irritation. If eye contact occurs, flush eye(s) with water for 15 minutes and consult a physician.

Keep out of reach of children.

Flammable. Do not use near an open flame or while smoking.

Information

Prep Barrier Film provides a visible, barrier film of protection when applied to the skin prior to application of an ostomy appliance.

Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Do not re-use



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



Recyclable packaging