Freedom® Cath

Freedom® Active Cath

en Urisheath

es Vaina urinaria

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Instructions for use / Instrucciones de uso / Mode d'emploi

23327340 Version 1

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © El logotipo de Coloplast es una marca registrada de Coloplast A/S. Todos los derechos reservados. © Le logo Coloplast est une marque déposée de Coloplast A/S. Tous droits réservés. © 2021-03-23

en

Intended purpose

Urisheaths are intended to drain urine, normally into a urinary collection bag.

Indications

Urisheaths are indicated for male patients suffering from urinary incontinence.

Warnings

Re-use of the single use product is not recommended as cross contamination may occur.

Reprocessing, washing, disinfection and/or (re)sterilization may compromise product characteristics, causing additional risk of physical harm or infection to the user.

This product contains natural rubber latex which may elicit allergic responses in individuals who are sensitised to latex.

Cautions

If you experience a suspected allergic reaction or any other side effects, please contact your healthcare professional.

Information

This is a single use product.

The urisheath is self-adhesive.

The device does not have to be removed before a MR scan as it is MR safe.


Do not use with alcohol-based skin preparations unless thoroughly dry. Residual alcohol may cause degradation of material or excessive adhesion to skin.

Do not use with creams and powders. Using these products with the urisheath could stop the adhesive from working effectively.

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

How to use


Preparation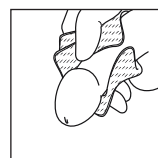


Trim pubic hair if necessary. Wash the penis with soap and water. Dry thoroughly.

Application


When applying the product, leave the foreskin in place over the head of the penis to prevent tangling or pulling of the loose skin. This applies to uncircumcised men. If circumcised, this instruction does not apply.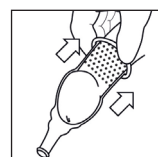

Place the rolled urisheath over the head of the penis with the narrow end pointing away from the body. Leave a 1-2 cm gap between the tip of the penis and the narrow outlet tube to allow the free flow of urine. Slowly unroll the product to its full length, or to the length of the penis.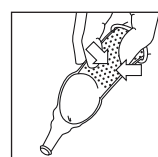


Gently squeeze the unrolled urisheath around the shaft of the penis to ensure adhesion to the skin and to seal off any wrinkles. No additional adhesive is necessary.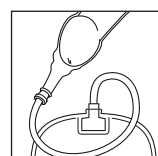

Attach the product to a urine collection bag.

The urinary collection system is now functional.

Removal

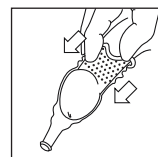Disconnect the product from the urine collection bag.

Grasp the urisheath at the base and roll forward gently.

Disposal

Change the product according to local guidelines and recommendations. Each product should be worn for no longer than 24 hours.

Skin maintenance: If possible, allow the skin to breathe for short periods between product changes.

The product is intended for single use only and should be disposed of in accordance with local guidelines, e.g. with normal household waste.

Do not flush the product down the toilet.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Explanation of symbols

Medical device

Indicates that the product is in compliance with European legislation for medical devices

Catalog number

Use-by date (YYYY-MM-DD)

Batch code

Date of manufacture (YYYY-MM-DD)

Manufacturer

Consult instructions for use

Do not re-use

Contains or presence of natural rubber latex

Indicates a barcode as containing Unique Device Identification

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