

Brava®



Skin Cleanser



Instructions for use

23327608

Version 1 The Coloplast logo is a registered trademark of Coloplast A/S.

All rights reserved. © 2021-04-23

Intended purpose

Brava Skin Cleanser is intended for cleaning intact skin within ostomy and continence care.

Warnings

Re-use of the single use product may create a potential harm to the user. Reprocessing, washing, disinfection and/or sterilisation may compromise product characteristics, causing additional risk of physical harm or infection to the user.

Cautions

For external use only.

Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

How to use

Open the package by pulling the re-sealable label. To enable re-sealing please do not remove the entire label.

Pull out one Brava Skin Cleanser wipe.

Use the wipe to clean the skin. Use as many wipes as are required.

Let the skin dry before applying the adhesive/appliance.

Re-seal the package to keep the Brava Skin Cleanser wipes moist.

Disposal

The product should be disposed of with normal household waste or according to local guidelines.

Do not flush the product down the toilet.

Composition

Polyethylene terephthalate/viscose (wipe), water, dexpanthenol, sodium citrate, citric acid, sodium gluconate, preservatives.

Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Indicates a barcode as containing Unique Device Identification



Global Trade Item Number



Keep away from sunlight



Recyclable packaging