

# Coloplast®

en Paste

Fills in cavities/  
folds

Ostomy Care



## Instructions for use

Please read the following instructions carefully.

23331836

Version 2

The Coloplast logo is a registered trademark of Coloplast A/S. All rights reserved. © 2024-03-11



## Intended purpose

The product is intended to fill in cavities/folds in the intact skin around the stoma. This provides a seal between the stoma and the ostomy baseplate and protects the skin against the stoma output.

## Cautions

This product is non-sterile and should be used with caution on premature babies or infants receiving incubator care as it may lead to increased risks of infection.

The product contains alcohol and should not be applied directly on to an open wound or skin ulcer. When applied to irritated or abraded skin a short stinging sensation may be experienced. Consult a healthcare professional if skin irritation occurs and persists.

Keep away from sunlight.

Store products at cool and dry conditions – do not freeze. Allow products to return to room temperature before application.

For external use only.

## Information

Coloplast accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to Coloplast's current recommendations.

## How to use


### Preparation

In order to avoid contamination of the paste


- Keep the tube away from water and prevent contact between the tube and stoma.
- Wash your hands with soap and water before and after applying the paste.

Ensure that the skin around the stoma is clean and dry before applying the Paste.


### Application

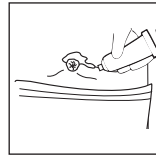

Apply the paste onto the skin to level deep skin folds, scars or uneven surfaces or apply directly on the adhesive/baseplate. Do not apply paste on the entire adhesive area.

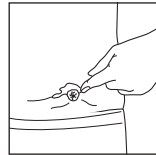

If needed, level the paste by gently smoothing using a spatula or finger that has been moistened.

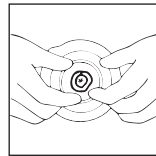

After applying the baseplate/adhesive, ensure a perfect fit around the stoma by pressing on the entire baseplate/adhesive with your fingers.

After use remove the residual paste on the tube with a clean tissue. Recap the tube tightly to prevent contact with moisture.

### Removal

The paste can easily be removed with lukewarm water and a wash cloth. An oily residue of paste can be left on the skin when applying the next appliance. It is better to leave a slight residue than risk irritating the skin through complete removal.

### Disposal

The product should be disposed of with normal household waste or according to local guidelines.

Do not flush the product down the toilet.

### Reporting of incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority.

## Explanation of symbols



Medical device



Indicates that the product is in compliance with European legislation for medical devices



Catalogue number



Use-by date (YYYY-MM-DD)



Batch code



Date of manufacture (YYYY-MM-DD)



Manufacturer



Consult instructions for use



Indicates a carrier that contains Unique Device Identifier information



Global Trade Item Number



Keep away from sunlight



Recyclable packaging