# Information to the patient with implanted device

- en** Folysil<sup>®</sup> LT catheters / Folysil<sup>®</sup> X-Tra catheters
- fr** Sondes Folysil<sup>®</sup> LT / Sondes Folysil<sup>®</sup> X-Tra
- de** Folysil<sup>®</sup> LT-Katheter / Folysil<sup>®</sup> X-Tra-Katheter
- it** Cateteri Folysil<sup>®</sup> LT / Cateteri Folysil<sup>®</sup> X-Tra
- es** Sondas Folysil<sup>®</sup> LT / Sondas Folysil<sup>®</sup> X-Tra
- pt** Cateteres Folysil<sup>®</sup> LT / Cateteres Folysil<sup>®</sup> X-Tra
- nl** Folysil<sup>®</sup> LT katheters / Folysil<sup>®</sup> X-Tra katheters
- el** Καθετήρες Folysil<sup>®</sup> LT / Καθετήρες Folysil<sup>®</sup> X-Tra
- pl** Cewniki Folysil<sup>®</sup> LT / Cewniki Folysil<sup>®</sup> X-Tra
- cs** Katétrý Folysil<sup>®</sup> LT / Katétrý Folysil<sup>®</sup> X-Tra
- fi** Folysil<sup>®</sup> LT -katetrit / Folysil<sup>®</sup> X-Tra -katetrit
- sv** Folysil<sup>®</sup> LT-katetrar / Folysil<sup>®</sup> X-Tra-katetrar
- da** Folysil<sup>®</sup> LT-katetre / Folysil<sup>®</sup> X-Tra-katetre
- no** Folysil<sup>®</sup> LT-katetere / Folysil<sup>®</sup> X-Tra-katetere
- bg** Катетри Folysil<sup>®</sup> LT / Катетри Folysil<sup>®</sup> X-Tra
- hu** Folysil<sup>®</sup> LT katéterek / Folysil<sup>®</sup> X-Tra katéterek
- sk** Katétre Folysil<sup>®</sup> LT / Katétre Folysil<sup>®</sup> X-Tra
- ro** Catetere Folysil<sup>®</sup> LT / Catetere Folysil<sup>®</sup> X-Tra
- sr** Folysil<sup>®</sup> LT kateteri / Folysil<sup>®</sup> X-Tra kateteri

REF.: SK3038

Date of issuance : March 2023

Revision number: 1

# en - Information to the patient with implanted device

## Folysil<sup>®</sup> LT catheters Folysil<sup>®</sup> X-Tra catheters

Madam, Sir,

You have been implanted with Folysil<sup>®</sup> LT catheter or Folysil<sup>®</sup> X-Tra catheter for treatment purpose. Please read carefully the below information for optimal follow-up.

### How to manage follow-up of my implanted device?

- It is recommended to maintain a good fluid intake .
- You have been supplied an implant card completed by your physician. This card mentions your name, your implantation date, your healthcare institution/provider and its address together with information on the implanted device and the manufacturing company.
- This card must be kept with you for the whole implantation duration to inform any other healthcare professionals of your implanted device.
- This catheter is purported to stay in your body for a certain while, from a few days to several weeks (maximum 12 weeks), depending on your disease and as prescribed by your physician. You are advised to not forget your removal date.
- You are advised to consult your physician or nurse for regular monitoring of your catheter as planned after the procedure to evaluate the treatment efficacy and check for any complication.

### What is Folysil<sup>®</sup> LT catheter or Folysil<sup>®</sup> X-Tra catheter?

Folysil<sup>®</sup> LT or Folysil<sup>®</sup> X-tra catheter is a thin, silicone hollow tube to drain urine from your bladder. It is equipped with a small balloon at the bladder side, and with several channels to allow drainage and instillation when needed.

The catheter is implanted within bladder either by natural urethral route or through a surgical incision in the lower abdomen (suprapubic route).

In both cases, the catheter is held in the bladder by the balloon inflated with sterile water or 10% aqua-glycerine solution, which prevents it falling out. Urine is drained from the bladder through the tube connected to a urine collection bag, which can either be strapped to your leg or attached to a stand on the floor.

“ Folysil<sup>®</sup> LT catheter” or “ Folysil<sup>®</sup> X-Tra catheter” is made of:

- Mix of silicone (min 98%) and peroxide (max 2 %) for the catheter tube and the balloon,
- Silicone (100%) for the distal tip
- Mix of silicone (70.5%), barium sulfate (26.2%), pigment (2%) and peroxide (1.3%) for the radiopaque line,
- Silicone glue.

### Which precautions should I take?

- Inform your physician in case of any allergy known to one of the above components.
- This device may act as a foreign body and be associated with inconveniences: some of them are mild and temporary like limited and transient pain or discomfort.
- In some cases, complications, or symptoms that could indicate that the device is malfunctioning, you are advised to immediately inform your physician or your nurse. The warning symptoms include: unusual pain or discomfort, cystitis symptoms, fever, bleeding, urine leakage, displacement

# en - Information to the patient with implanted device

## Folysil<sup>®</sup> LT catheters Folysil<sup>®</sup> X-Tra catheters

of the catheter, reduced urine flow or urinary retention.

- Don't kink the catheter tube nor the urine bag tubing.

- You should ensure scrupulous local hygiene, cleaning the genital and anal areas every morning and evening with soap and water. For suprapubic placement, wash the area around the catheter exit site gently with soap and water, every day.

### I have an implanted Folysil<sup>®</sup> LTcatheter or Folysil<sup>®</sup> X-Tra catheter . Can I safely pass medical imaging exams or airport security?

- In case of any medical, surgical or exploratory procedure, you are advised to inform the attending physician of your implanted device and show him your implant card.

- There is no contraindication to perform a MRI scan and no interference with any external equipment.

### Be aware that:

- This device is for single use and should not be re-used.

- After removal, this device is dirty and should be discarded according to the hospital policy or normal waste in case of removal at home.

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